



## Action Trackchair® Operating Instructions

### Overall Instructions:

- When transferring into the Trackchair®, make sure the controls are in the **OFF** position before sitting.
- When operating the Trackchair®, make sure you are securely fastened in with either the lap belt or 4-point harness.
- The Trackchair® control has five speeds, one – five, which can be changed with the up and down arrows.
- Optional lighting is available and is controlled on the joystick control panel.
- If it is necessary to pull the Trackchair®, disengage the brakes on the motors with the levers on the back of the motors. Push levers to the outside on both motors. Do not pull the Trackchair® over 5 MPH.
- Action Manufacturing does not recommend operating Trackchair® in salt water. Although the Trackchair® is powder coated to the highest quality with very durable powder coat, salt water is very corrosive and can cause problems with powder coat and metal.

### Safety Instructions:

- Do **NOT** ride the Trackchair® during loading or unloading from vehicle or trailer.
- Only one person should be on the Trackchair® at any time.
- A seatbelt is required when operating the Trackchair®.
- Keep hands and fingers away from the rotating parts of the tracks and pinch points.
- Make sure controls are turned **OFF** when getting in or out of the seat.
- Make sure controls are turned **OFF** before placing a cover over the Trackchair®.
- When approaching inclines or declines in any surface always use caution, switch to a slower speed.
- **USE CAUTION:** The Trackchair® will climb inclines enough to tip over in any direction.
- Do **NOT** attempt to climb stairs in the Trackchair®.
- When climbing over obstacles, approach at an angle, **NOT** perpendicular.
- Do **NOT** navigate Trackchair® on more than a 20-degree slope.

**FAILURE TO KNOW THE LIMITS AND TO ADHERE TO THESE INSTRUCTIONS COULD CAUSE PERSONAL INJURY OR EQUIPMENT DAMAGE**

### Comfort Adjustments:

- The footrest can be moved up or down to fit the rider's needs. The chair itself can be leveled to the desired comfort of the rider. The armrest can flip back for easier transferring into the chair.
- Trackchair® will tilt forward and backward by depressing the rocker switch on the grab handle.
- Trackchair® arms will move up and down by pulling the pin on the rear of arm and lifting.

### Returning Trackchair®:

- Once battery level reaches half charge, return to training site. Three LED lights indicate there is only 50% battery remaining. It is required that the rider return to training site when battery indicator reads as pictured below:



- Trackchair® must be returned to training site by 5 p.m., regardless of reservation start time.